




Patella stabilisation

Reconstruction of Medial
Patellofemoral Ligament (MPFL)
with **Infinity-Lock™** 5 mm Tape


Safe
Strong
Simple

Why **Infinity-Lock™** 5 mm Tape for the reconstruction of MPFL?

- Stable, safe solution to MPFL laxity
- No need for autograft or allograft
- No donor site morbidity
- Promotes tissue ingrowth
- Improved patient outcomes*

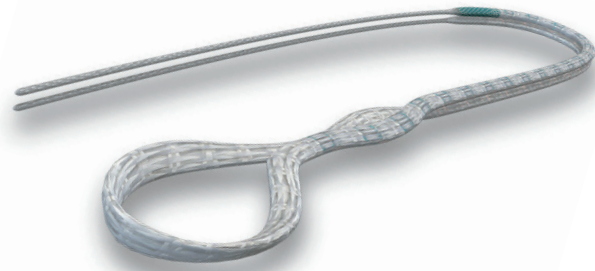

How does it work?

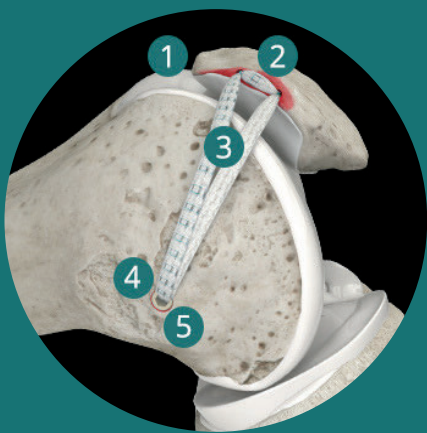

1. Make two small incisions, medial border of patella and at femoral attachment point
2. Uses two all-suture anchors spaced along a shallow sub-cortical trough on the medial patella border
3. Two legs of Infinity-Lock tape are passed from the anchors to the attachment point, creating a double-bundle
4. Attachment point and tension checked before fixing
5. Securement on femur is with 6 mm interference screw

Order code

Infinity-Lock Tape (supplied sterile):

102-1098 Infinity-Lock Tape, 5 mm x 400 mm

Want to know more?



Follow the link for surgical technique video, animation, clinical data and more.

Get involved

At Xiros we are constantly striving to improve outcomes for patients, and we want to hear from you. Help us create the future of sports medicine and share your opinions and expertise with us at: www.xiros.co.uk

* Deo, H., R. Mohamed and G. Ahmed (2023). "Medium-term outcome of medial patellofemoral ligament reconstruction using synthetic graft." Knee 44: 220-226.