

Introduction

Data from recent publications suggest that the **Pitch-Patch** for augmentation of massive RC tears feasibly reduces the retear rate and significantly improves functional, structural, and clinical outcomes over the long term.

Publications

Dommer et al, 2023

- Data were collected retrospectively on 15 patients who underwent open or arthroscopic patch augmented RC repair using the Pitch-Patch.
- Good muscle quality (Goutallier grade $\leq$ II), high retraction (Patte grade $\geq$ 2) and short tendon length ($<$ 15 mm) were required to include patients in the study.
- Mean follow-up was 43.8 months.
- Functionally the outcomes improved significantly.
- Overall satisfaction with surgery was 87%, with the same number stating they would have the surgery again.
- The structural failure rate, described as Sugaya grade 4 and 5, was 53%, showing a favourable comparison to the previously reported rate of 92% for this population with short tendon length.
- Only 2 patients had any complaints about pain at the last follow-up with visual analog scale score of 2 and 1 of 10, respectively.
- Using the Pitch-Patch as a scaffold to strengthen the fixation gives the reconstruction more stability for an increased chance of healing and thus lowers the structural failure rate compared to nonaugmented repairs in presence of a good muscle quality and short tendon length.

Table 1. Results Summary for Dommer et al (2023)

	Preoperative (mean, SD)	Follow-up (mean, SD)	P Value
Constant-Murley score (relative)	36.1 (14.2)	84.3 (7.3)	$< .01$
Pain (visual analog score)	5.3 (3.3)	0.2 (0.6)	$< .01$
Subjective Shoulder Value	34.7 (20.2)	89.5 (12.1)	$< .01$
ROM (degrees) – Forward elevation	111.0 (40.2)	167.5 (6.2)	$< .01$
ROM (degrees) – Abduction	85.7 (28.8)	153.3 (22.7)	$< .01$
ROM (degrees) – External rotation	33.3 (21.9)	41.3 (20.9)	$< .090$
Abduction force (kg)	0.7 (0.4)	6.5 (2.9)	$< .01$
Tear size (mm)	33.3 (5.7)	17.3 (10.6)	$< .01$

Smolen et al, 2020

- Data were collected prospectively on 50 patients with massive RC tears involving at least 2 tendons who underwent arthroscopic RC reconstruction using the Pitch-Patch.
- Mean clinical midterm and final follow-up was 22 months and 52 months respectively.
- Clinical outcomes at midterm and final follow-up including all subcategories of the CS and the SSV improved significantly from preoperative values.
- Radiologic midterm follow-up showed an intact reconstruction with complete footprint coverage in 86% of all tendons (43 patients).
- Only 14% (50 patients) experienced a rerupture, and only one of these cases involved a rupture of the patch.
- Histologic analysis of one intact patch explanted due to symptomatic crepitus revealed no signs of foreign body rejection. Additionally, it was observed that the patch was covered in a fibrous, almost bursal-like mantle of tissue.
- Six patients (12%) with intact reconstructions were revised for frozen shoulders or arthrofibrosis, however after revision surgery, all of them showed comparable good clinical results with the patch in-situ.

Table 2. Results summary for Smolen et al (2020)

		Preoperative (mean, SD)	Midterm follow-up (mean, SD)	Final follow-up (mean, SD)
SSV		40.3 (24.3)	89.2 (12.9)	89.6 (15.2)
CS Subjective	Pain	5.5 (3.1)	13.6 (2.0)	14.2 (1.7)
	ADLs	10.3 (3.6)	19.4 (1.0)	19.4 (1.4)
CS Objective	Flexion	6.2 (2.8)	9.9 (0.4)	9.8 (1.0)
	Abduction	5.2 (2.9)	9.6 (1.2)	9.6 (1.3)
	External rotation	3.6 (3.8)	9.8 (1.0)	9.8 (1.2)
	Internal rotation	3.8 (2.2)	7.1 (1.7)	8.0 (1.8)
	Strength	1.9 (3.1)	11.9 (6.0)	12.6 (6.4)

Hess et al, 2023

- This prospective study continues tracking the same cohort of 50 patients published by Smolen (2020) to compare complete follow-up data measured at 3, 12, and 60 months after surgery.
- Clinical outcomes (CS and SSV) showed significant improvement from baseline at the 3-month follow-up, as well as between the 3-month and 60-month follow-ups, demonstrating continued clinical improvement over the long term.
- More favourable results were observed at the longest follow-up visit, which indicates healing was achieved and sustained in most patients.
- No new or worsening of previous reported complications were detected during this extended follow up period.

Table 3. Results Summary of for Hess et al (2023)

	Preoperative (mean, IR)	3 months follow-up (mean, IR)	1 year follow-up (mean, IR)	5 years follow-up (mean, IR)
SSV	32 (24-47)	80.5 (73.5-88.5)	84 (76.5-90)	85 (81.5-91.5)
CS	40 (20-50)	90 (80-99)	95 (82.5-100)	95 (85-100)

Conclusions from the recent studies

- Patch augmented cuff repair using the Pitch-Patch leads to a significant improvement of functional and structural outcomes.
- Rotator cuff repair using the Pitch-Patch achieved good clinical outcomes over the long term.
- Data suggested that the retear rate of 14% compared favourably with nonaugmented RC repairs in the literature.
- The retear rate of 53% in patients with short tendon length is favourable compared to the previously reported rate of 92%



Pitch-Patch Clinical Summary

Compilation of publications for Rotator Cuff tears using the Pitch-Patch

Abbreviations

ADL: Activities of Daily Living
CS: Constant-Murley score
IR: Interquartile Range
RC: Rotator cuff
ROM: Range of Motion
SD: Standard Deviation
SSV: Subjective Shoulder Value

Study Citations

Dommer, L. M., et al. (2023). "Massive rotator cuff tears with short tendon length can be successfully repaired using synthetic patch augmentation." J Shoulder Elbow Surg 32(10): 2089-2096.

Smolen, D., et al. (2020). "Application of a new polyester patch in arthroscopic massive rotator cuff repair-a prospective cohort study." J Shoulder Elbow Surg 29(1): e11-e21.

Hess, F., et al. (2023). "Long-Term Outcomes Following Synthetic Patch Augmentation to Treat Rotator Cuff Tears." Trauma International 9(1)

Xiros™ Ltd

Springfield House, Whitehouse Lane, Yeadon, Leeds LS19 7UE

T. +44 (0) 113 238 7202

E. enquiries@xiros.co.uk

Xiros Limited, registered in England No. 1664824

All rights reserved. © Xiros™ 2026.

Worldwide patents and patents pending.

WP 039 2.00